Lovey

Pacifier & Teether Holder

Lovey Contents: Polyester Fiber

Lovey Cleaning:

Machine wash cold. Do not bleach. Tumble dry low. Inspect all components of the Lovey before each use. Watch for wear on the feet and straps. If any damage is detected discontinue use.

Pacifier Cleaning:

To ensure hygiene, boil for five minutes before first use. Wash prior to each use. Safe for use in dishwasher (top rack only), electric and microwave sterilizers, or boiling water.

WARNING – Do Not Tie Pacifier Around Child's Neck as it Presents a Strangulation Danger.

For your child's safety

WARNING!

Inspect carefully before each use. Pull the pacifier/soother in all directions. Throw away at the first signs of damage or weakness.

Only use dedicated pacifier/soother holders tested to EN 12586. Never attach other ribbons or cords to a pacifier/soother, your child may be strangled by them.

Do not leave a child unattended with a pacifier/soother. Do not use as a teether. Check for signs of cracking, traces of bites, tearing, or other damage, especially when the child develops teeth. Small pieces can accidentally be inhaled and cause serious injury. In the event that the pacifier/soother becomes lodged in the mouth, DO NOT PANIC; it cannot be swallowed and is designed to cope with such an event. Remove from mouth with care, as gently as possible. Do not leave in direct sunlight, near a source of heat, or leave in a sterilizing solution for longer than recommended as this may weaken the pacifier/soother. To avoid tooth decay, never dip nipple/teat in sweet substance or medication. Replace between one and two months of use for safety and hygiene reasons.

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